

# DUAL LUMEN EXTENDED LENGTH CATHETER


*Designed for use with the Aquadex SmartFlow™ System*



- Two independent 16G lumens
- 6F reverse taper for smooth insertion
- Coil reinforcement for enhanced kink resistance


# Dual Lumen Extended Length Catheter (dELC)

The dELC is a 6F reverse taper dual lumen venous access device with stainless steel coil reinforcement, designed specifically for use with the Aquadex SmartFlow system. It has female luer connectors and offset tip.

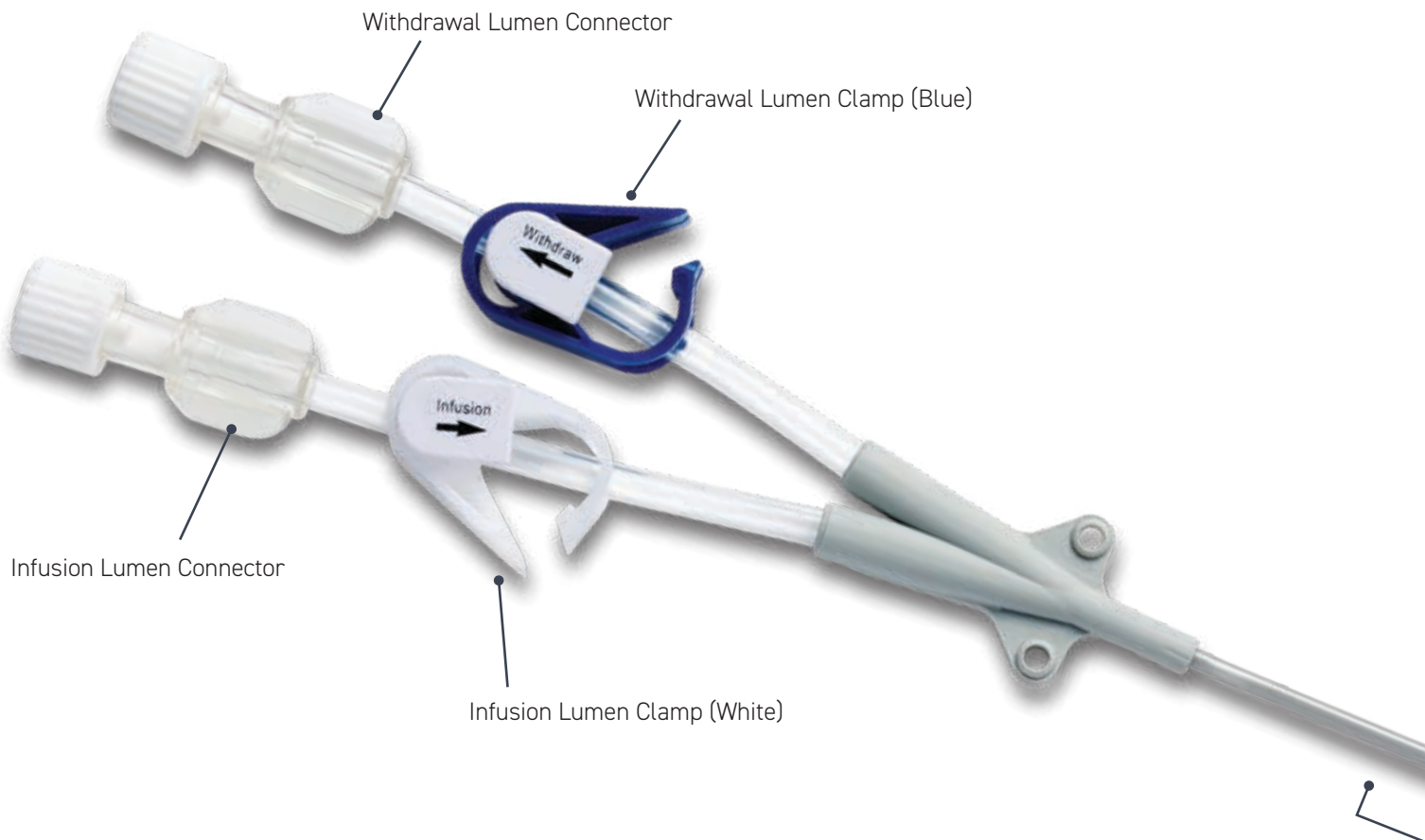

## dELC DETAILS


Lumen	Clamp Color	Distal Opening	Guidewire Insertion	Acceptable Guidewire Diameter	Approximate Priming Volume
Withdrawal	Blue	At approximately 0.5 cm proximal to distal catheter tip (offset lumen)	No	Use Infusion Lumen	0.75 ml
Infusion	White	At distal tip of catheter	Yes	Up to 0.018" (0.46mm)	0.75 ml

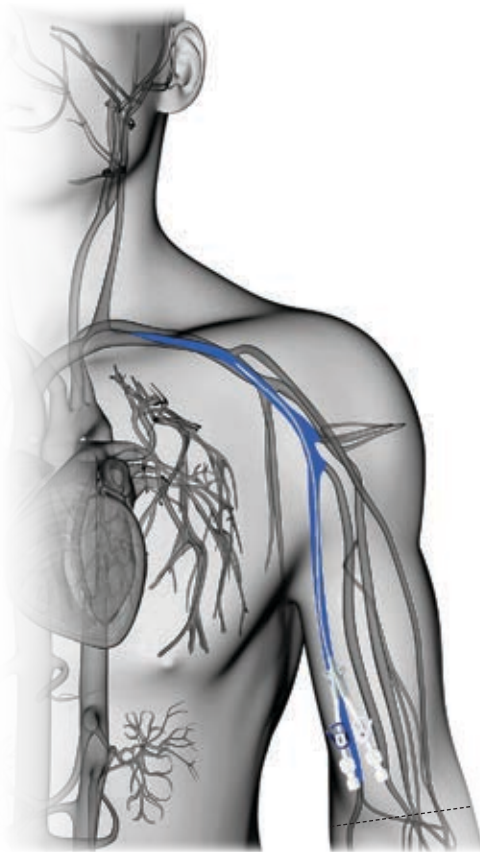

Antecubital Fossa

## Recommended Catheter Placement

The catheter is inserted into a peripheral vein. The recommended vein is the basilic. Placement of the catheter above the Antecubital Fossa is recommended.

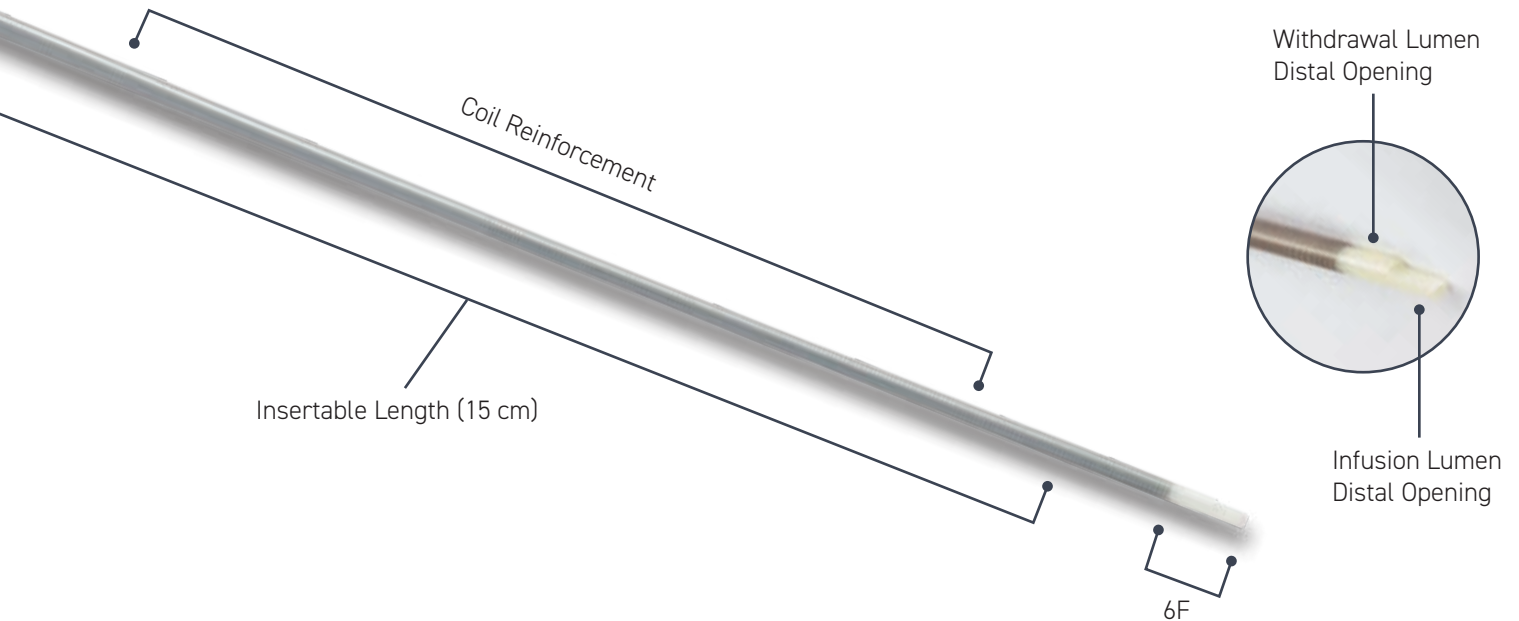

# DUAL LUMEN EXTENDED LENGTH CATHETER

*Designed for use with the Aquadex SmartFlow™ System*

The Aquadex SmartFlow System is indicated for: Continuous ultrafiltration therapy for temporary (up to 8 hours) or extended (longer than 8 hours in patients who require hospitalization) use in adult and pediatric patients weighing 20 kilograms or more whose fluid overload is unresponsive to medical management, including diuretics. All treatments must be administered by a healthcare provider, within an outpatient or inpatient clinical setting, under physician prescription, both of whom having received training in extracorporeal therapies.

## ORDERING DETAILS

Model	Product Description	Size	Quantity
114153	Dual Lumen Catheter with Coil Reinforcement	6F (.070" / 1.78mm I.D.)	1 Box of 5
115463	Catheter with Insertion Kit	6F (.070" / 1.78mm I.D.)	1 Box of 5 Each



Overall Length Including Hub and Extension Tubing: 11" (28cm)

Outside Diameter (OD) at Hub: 0.092" (0.23cm)

Outside Diameter (OD) at Distal End: 0.081" (0.20cm)

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.

Aquadex SmartFlow™ is a trademark of CHF Solutions, Inc.

**chf solutions**

**CUSTOMER SUPPORT**

1-855-786-2778

Support@CHF-Solutions.com

www.chf-solutions.com